



	BUTTONS	INFUSIONS	SCREEN
PUMP ON		• Bolus dose delivery.	
	and depress simultaneously	• Bolus interruption (if in execution).	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
PUMP ON	1 <sup>st</sup> depressing	• Administered flow rate in ml since start of infusion.	
	2 <sup>nd</sup> depressing	• Administered patient bolus in ml since start of infusion.	
	3 <sup>rd</sup> depressing	• Administered number of bolus.	
	4 <sup>th</sup> depressing	• Administered clinician bolus in ml since start of infusion.	
	5 <sup>th</sup> depressing	• Total administered volume in ml since start of infusion.	

	BUTTONS	SCREEN REVIEW OPTIONS	SCREEN
PUMP ON	1 <sup>st</sup> depressing	• Programmed flow rate.	
	2 <sup>nd</sup> depressing	• Programmed bolus dose.	
	3 <sup>rd</sup> depressing	• Programmed interval time between boluses.	
	4 <sup>th</sup> depressing	• Programmed number of boluses per hour.	
	5 <sup>th</sup> depressing	• Programmed partial volume.	
	6 <sup>th</sup> depressing	• Programmed clinician bolus dose.	

CRONO five

AUG 01/USA/00 Crono FIVE 07/05

	BUTTONS	CLINICIAN BOLUS FUNCTION	SCREEN
PUMP OFF		• Press for 7 seconds.	
	press without releasing  button	• Key-pad lock level selection.	
		• Select clinician bolus level.	
	/	• The screen will show the latest administered clinician bolus dose.	
	/	• Decrease/Increase clinician bolus	
PUMP ON		• The screen will show that the clinician bolus is available.	
	and  depress simultaneously	• Clinician bolus delivery.	
	and  depress simultaneously	• Clinician bolus function interruption.	

After the administration of one clinician bolus, the lock level L 2c switches automatically back to the lock level programmed when the clinician bolus procedure started i. e. either L 0 or L 1 and no more clinician bolus can be delivered without repeating the clinician bolus procedure from the beginning.

	BUTTONS	PUMP OFF	SCREEN
PUMP ON	and  depress simultaneously	• Pump switches OFF.	

	BUTTONS	END OF INFUSION	SCREEN
END OF INFUSION	and  depress simultaneously	• End of infusion. • Piston automatic reversing. • Automatic switch OFF.	

Intra Pump Infusion Systems 920 Minters Chapel Rd, Suite 200, Grapevine, TX 76051  
Toll Free: 1-866-211-7867(pump) Fax: 630-845-2768 www.intrapump.com

Man.: Canè S.r.l. Medical Technology Rivoli (TO) - Italy